

Prof. Dr. med. Klaus Stelter Beratung für Mediziner Eppanerstr. 5

Eppanerstr. 5 83024 Rosenheim www.medwebdesign.de

Dr. med. K. Steller, Eppanerstr. 5, 83024 Rosenheim
An Frau
Heike Jordan
Meyer-Haake GmbH
Daimlerstr. 4
61239 Ober-Mörlen

Rosenheim 18.01.2019

Evaluation radioSURG® 2200

Dear Mrs. Jordan,

Many thanks for giving me the opportunity to test your radioSURG 2200[®] radio frequency generator including the probes for ENT and plastic surgery.

I was able to test the RF generator during inpatient and outpatient operations in the hospital and my private practice and I am very convinced of your system. Meyer-Haake has come a little closer to finding the "all-in-one device suitable for every purpose". The radioSURG® replaces all RF and HF electro surgical applications as well as many laser applications. The touch screen is intuitive and easy to use and readable from all sides, even in bright surgical light. The distinction between cutting procedures (yellow plugs and sockets) and interstitial, coagulating procedures (blue plugs and sockets) also makes operation easy for the operating room staff.

The settings are immense and have been tested by myself in many situations. If the surgeon has found his preferred setting, he can save it and create his own profile. This allows easy and quick changeover between several surgeons. Whereby the most innovative operating aid is the foot pedal. Here, the sterile surgeon standing at the table can autonomously switch back and forth between the different modes and power outputs without having to rely on external help (surgical stand-by man).

The supplied probes are very thin and are available in all possible angles and lengths. This makes fine skin incisions and targeted interstitial thermal procedures safe and easy. Of course, the very fine probes are more sensitive opposite to bending and frequent sterilizing cycles, here replacement should be arranged in time. I have successfully used the radioSURG® for the following procedures:

- reduction of the turbinates
- tightening of the soft palate
- tonsillar surgery
- reduction of the lingual tonsil
- blepharoplasty
- face-lift and parotid skin cut
- wrinkle treatment (RF-ReFacing®)
- throat and neck surgery (neck dissection, thyroid, neck cysts)
- Tumor excisions in the mucosal area

Due to the very low penetration depth of the coagulation, very precise incisions with good histopathological assessability could be produced, which also qualifies the radioSURG® for oncological interventions.

The Company Meyer-Haake offers an excellent, fast service and commits themselves to individual needs of the surgeons. Even custom-made instruments are possible and will be realized promptly. The development of new probes or surgical procedures are gratefully accepted and implemented.

This unbureaucratic speed is what I miss when working with the large providers and what makes working with the family business absolutely recommendable.

I hope I could help you with my evaluation.

Kind regards and many thanks

Prof. Dr. med. K. Stelter